

Absorbable Hemostat Gelatin Powder



PAHACEL® Gelatin Hemostat is an absorbable gelatin powder for hemostatic use on bleeding surfaces. PAHACEL® Gelatin Hemostat is beige in appearance and a gamma sterilized powder.

- Easy to prepare a puttylike paste with special stirring cup!
- Provides the fast hemostasis within 2 minutes

Easy to Use

STOP BLEEDING

- Open the vial and pour the powder in stirring cup.
- Add the saline solution 4-6mL (desired) into powder and knead.
- After puttylike paste formed, it can be use directly or via syringe to desired surface.

How to Supply

ABSORBABLE HEMOSTAT	
Pahacel Gelatin Powder	
Reference	Size
PCGP28	1 g
PCGP29	2 g

